

IGENITY[®] HAIR SAMPLE INSTRUCTIONS

1. Pull (do not cut) hair from the tail switch, not tail head.



2. Make sure hair is free of fecal material and dirt. Roots must be clearly visible. Approximately 20 to 30 hair roots are needed.




3. Write the animal's ID number on the collector in the space provided.



4. Open the collector and peel back the plastic cover.



5. Insert the root end of the hair in the middle of the collector. Close the collector by sealing the plastic cover over the hair and then insert the tab into the slot.

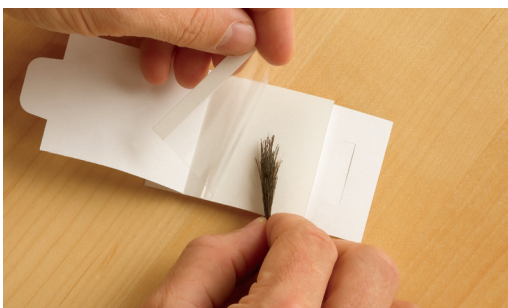

6. Trim the excess hair extending from the collector. Put all collectors in a plastic bag for shipment.



4131 N. 48th St. Lincoln, NE 68504 • 1/877-IGENITY • 402/435-0665
geneseekinfo@neogen.com • www.neogen.com/genomics